

TECHNICAL DATA SHEET - PROVISIONAL

NTT SOLID ELECTRODES V 3.1 - WITH NON CONDUCTIVE FABRIC




NTT silicone electrodes V 3.1 (With or without integrated cable) are distinguished by their long life, simple application and high performance due to **low resistance value** with **Improved conductive silicone recipe**. Additional protection for integrated Cable in V 3.1 makes stronger connectivity and provides electric current leak proof ability.

The product portfolio covers numerous solutions such as medicine, sports and leisure activities, for the measurement of electrocardiography (ECG), EMG, training monitoring, electro-muscle stimulation (EMS) and more.

The simple transfer technology options allows an optimum and permanent bond between the NTT electrode and the textile. Various shapes, varieties (with or without integrated cable) and personalised product designs are available through out the range. Tailored to the human skin impedance, the Biocompatible System enables the transmission of an optimal electrical signal.

SensElast Connect
Examples of available sizes & forms

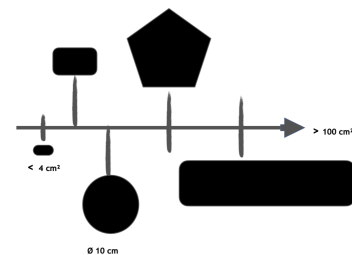

MATERIAL INFORMATION

Nr.	Material Type	Description	Color
1.	Fabrics	Non conductive knitted fabric	Black
2.	Basis	Conductive silicone rubber - Improved Recipe - E	Black
3.	Adhesive tape (If applicable)	PU Thermoadhesive foil	Transparent
4.	Connection cable (If applicable)	Insulated Cable	
5.	Upper and lower link connectors (If applicable)	Silver conductive knitted fabric fused with Theremoadhesive foil	Silver


ELECTRODE TECHNICAL INFORMATIONS

Product performance range (resistance Ω) Note : surface to cable resistance - 2 point prob method	< 200 Ω /cm ²		
Thickness of electrode	0,9 mm - 1,5 mm		
Weight	0,09 g/cm ² $\pm$ 3 %		
Apperance	Matte - Standard available		
	Gloss - Contact for further details		
Elongation / Elasticity	Max. 2 %		
Application parameters of heat press (If applicable) Note : Given parameters are based on the Thermoadhesive foil. Parameters may have to be adjusted based on the machine type, base material.	Temperature range:	140 - 145 °C	
	Duration range:	20 - 25 second	
	Pressure range:	4,5 - 6 bar	
	10,000 cycles		

COMPLIANCE AND CERTIFICATION

Wash Test	: 50+ washes (Complete Electrode)
Skin sensitisation	: No skin irritation based on initial testing - Further testing in progress
Bio compatibility	: Testing in progress

ASSEMBLING GUIDELINE FOR ELECTRODES WITH CABLE

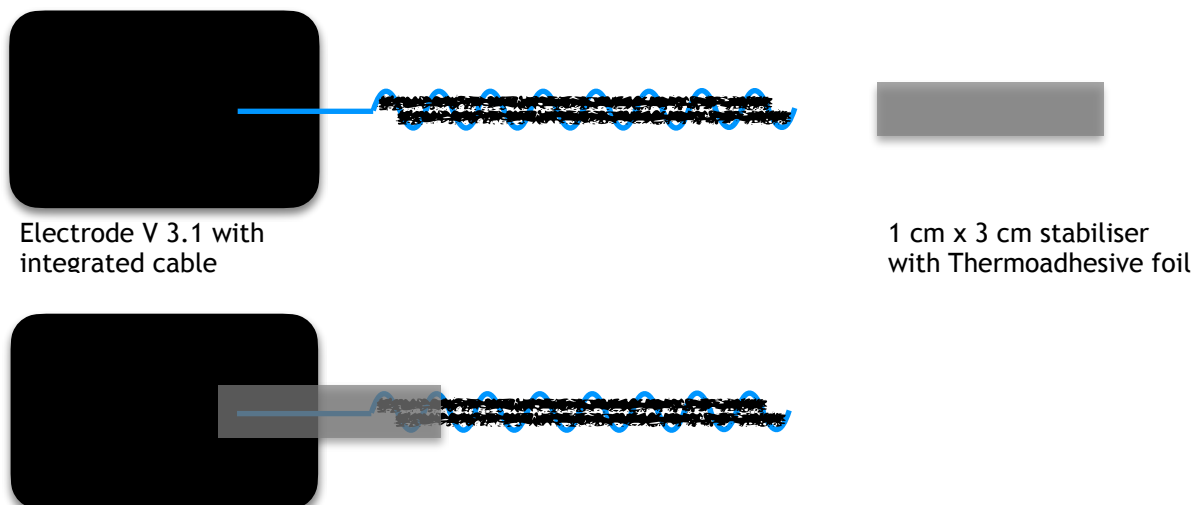

Electrode V 3.1 with integrated cable


1 cm x 3 cm stabiliser with Thermo adhesive foil

- Remove the paper backing under the electrode and also the 1 cm x 3 cm stabiliser

- Place the stabiliser mesh side towards the Electrode back side, covering the cable (Refer the image)
- Bond the electrode along with the stabiliser to the product as necessary

Important : The integrated stabiliser enhances structural integrity, ensuring prolonged service life and consistent performance under high-stress operating conditions.

ASSEMBLING GUIDELINE FOR ELECTRODES WITHOUT CABLE

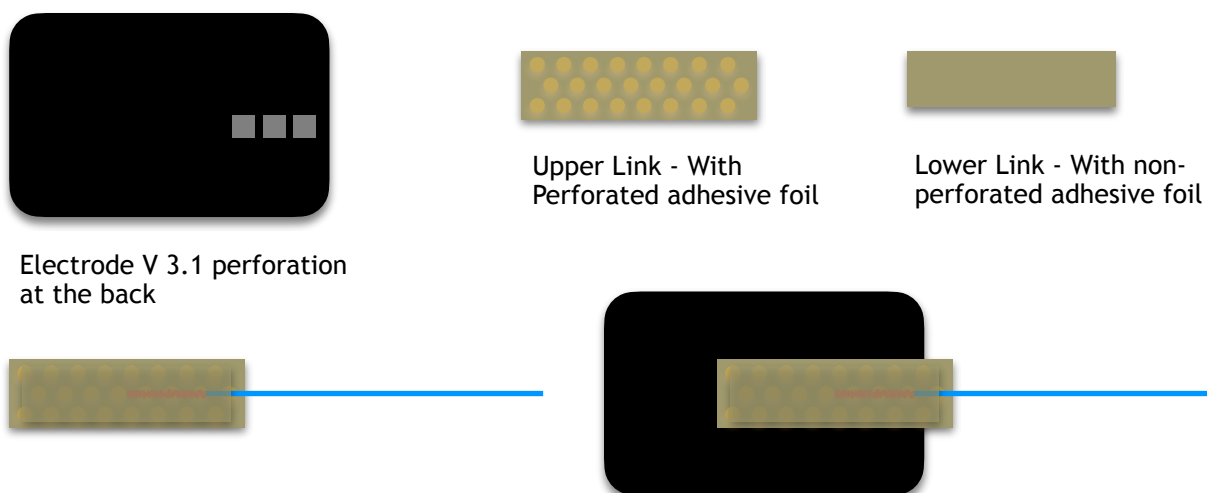

Electrode V 3.1 perforation at the back

Upper Link - With Perforated adhesive foil

Lower Link - With non-perforated adhesive foil

- Remove the paper backing under the electrode and also the upper and lower links
- Cable is placed between Lower link (non adhesive side) and Upper Link (adhesive side)
- Upper link (Non adhesive side) to be aligned with the Electrode perforation at the back

CONTACT INFORMATION

New Textile Technologies GmbH
Stollenau 10, D-72336 Balingen
info@ntt-int.com
+49 (0) 74 33-99 59-0

<http://www.ntt-int.com>